



Enterprise Imaging Actionable Findings

Two-way communication closes the loop

That's life in **flow**.

AGFA 
HealthCare

Actionable findings are observations that you wish someone to act on. There is a wide range of actionable findings, from critical ones that have high priority, to incidental findings, that only require follow-up in a couple of weeks or months

Critical findings are often handled well, in the sense that the radiologist calls the ordering clinician directly to ensure the patient gets the appropriate care or treatment. For less critical findings however, there is increasing evidence that there still a gap to be covered in closing the loop between the radiologist, the clinician and the patient. A systematic review reported that 1% to 36% of radiology results are not communicated, which can lead to missed diagnoses and worse.

The Enterprise Imaging Actionable Findings module allows to close the loop between radiologists and clinicians through rapid and effective communication of incidental and critical findings.



Up to 36% of radiology results are not communicated, which can lead to missed diagnoses and worse.

Enterprise Imaging Actionable Findings

- Improves the effectiveness of communication among caregivers.
- Allows escalating to fellow clinicians if the finding doesn't get acknowledged in time
- Documents in Enterprise Imaging which findings have been communicated to which clinician.
- Allows the radiologist to track the status of each finding within Enterprise Imaging
- Fits seamlessly into the radiologists' workflow.
- Supports healthcare providers to comply with Joint Commission requirement

Seamless two-way communication between radiologist and clinician


No matter how quickly and accurately the radiologist diagnoses a finding, if the clinician doesn't see it in time, it won't help the patient. You want to make sure the clinician not only receives the findings requiring attention, but also takes action on them. This is especially true for unexpected or incidental findings, which the clinician is not expecting.

Enterprise Imaging Actionable Findings makes both communication and acknowledgement a seamless part of the radiology workflow.



When the radiologist diagnoses a critical finding:

- 1 Either manually or automatically, the radiologist creates a notification for the ordering physician that special attention is required
- 2 Enterprise Imaging Actionable Findings sends the notification via SMS or email to the ordering physician or any other physician indicated on the order
- 3 To access the report and images, the ordering physician clicks the hyperlink in the message they receive. This triggers an automatic acknowledgement, sent to the radiologist
- 4 If the ordering physician doesn't click the hyperlink within a predefined period of time, the notification is escalated and automatically sent to another (group of) physician(s). The radiologist can also create lists with the status of all the reports and tasks that have been acknowledged or that are still pending, for true peace of mind

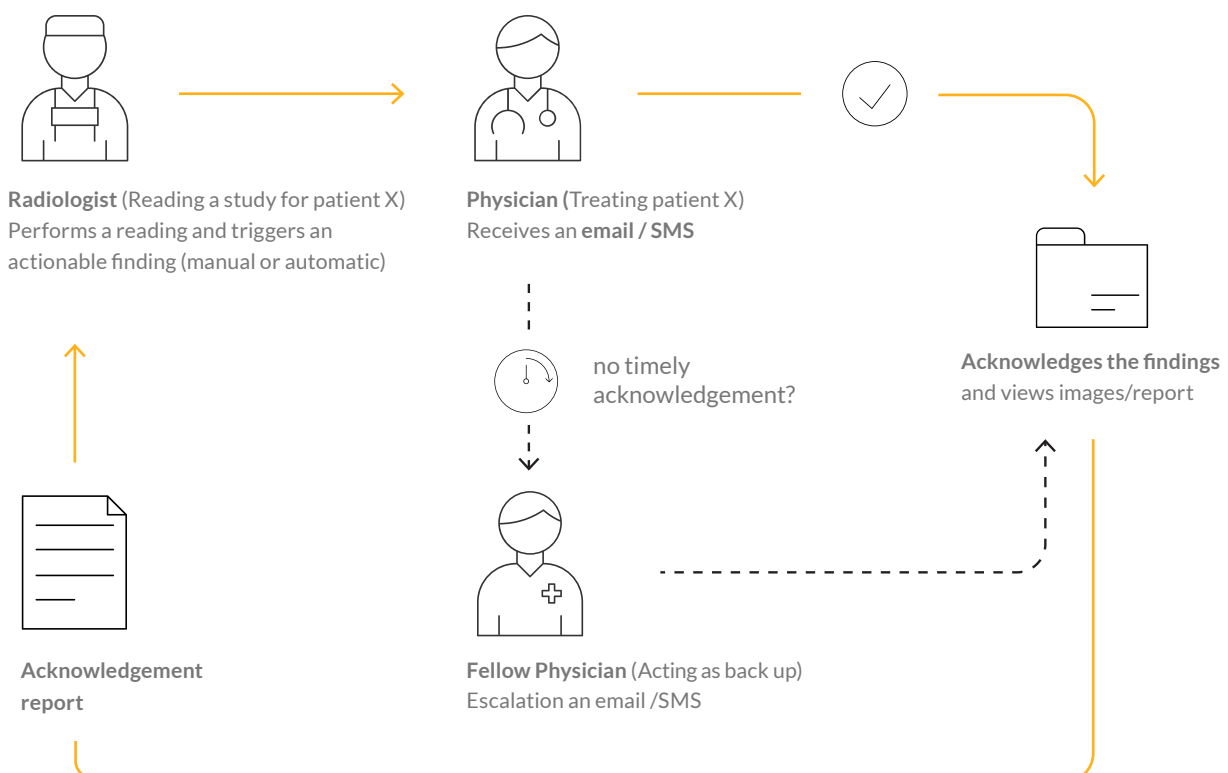

Manual communication

Enterprise Imaging Actionable Findings allows radiologists to manually create a notification right in the Enterprise Imaging workflow: no need to open up an email client or another program.

From predefined lists, the radiologist indicates the severity level, specialism and type of finding (as defined by ACR). They can also have the notification and link sent to additional physicians. A free text field allows them to add any notes or comments.

Automated notification

Enterprise Imaging Actionable Findings can also be flexibly configured to send out notifications of critical findings automatically. These can be triggered by predefined keywords or phrases, or based on the report severity, the procedures involved, the ordering physician, etc.

“All of the indicated clinicians receive the notification by email or SMS. The message contains a hyperlink which they can click to view the images and report directly in the XERO® Universal Viewer.”



Centralized follow-up and event logging

With Enterprise Imaging Actionable Findings, all cases are logged in one, centralized place. This enables the system administrator to easily follow up on any issues or retrigger findings if needed. They can also configure the escalation flow.

Contact your AGFA HealthCare Client Executive to get started

or email enterpriseimaging@agfa.com



AGFA, the AGFA rhombus, and the AGFA HealthCare logo are trademarks of Agfa-Gevaert N.V., Belgium, or its affiliates. The Enterprise Imaging Platform logo, GRIP, GRIP Services, Imaging Health Network, Imaging Health Record, IMPAX, RUBEE, That's life in flow, XERO, and XERO Viewer are trademarks of AGFA HealthCare N.V., Belgium or its affiliates. All rights reserved. All other trademarks are held by their respective owners and are used in an editorial fashion with no intention of infringement. The data in this publication are for illustration purposes only and do not necessarily represent standards or specifications that must be met by AGFA HealthCare. All information contained herein is intended for guidance purposes only, and characteristics of the products and services described in this publication can be changed at any time without notice. Products and services may not be available for your local area. Please contact your local sales representative for availability information. AGFA HealthCare diligently strives to provide as accurate information as possible but shall not be responsible for any typographical error.